

AlphaTRAK LANCETS

Ultra Thin Design for Less Painful Testing

100 Sterile Lancets, 28 gauge

For use with **AlphaTRAK® Blood Glucose Monitoring System** from Abbott Animal Health.

Directions:

Wash hands with soap and warm water. Dry well.

1. Load lancet into lancing device and twist off protective cap.
2. Choose a spot on the ear flap of cats, or on the side of a foot pad, or just inside the lip of a non-aggressive dog. Select a different site each time you take a blood sample.
3. Follow your veterinarian's instructions for obtaining a blood sample.

Safely discard the lancing device once used. Eject lancet from lancing device and snap-on safety cap before discarding.

To minimize the pain of testing: Change lancet after every use to ensure the least painful puncture. Lancets become dull and painful when re-used.