

Needles for Dogs, Cats & Horses

Hypodermic Sur-Vet Needles Designed For The Veterinary Market

Featuring a translucent hub and regular bevel, Terumo needles are known worldwide for extreme sharpness. They are ground and honed with a unique double-bevel design to provide a comfortable injection. Thin wall needles provide more comfort with an increased flow. Regular wall needles are less likely to bend.

Sterile, single use hypodermic needles. Silicone coated for easy penetration. Terumo needles are a favorite to use for subcutaneous fluid administration. Latex free.

Directions:

To assure a tight fit, push and twist the needle clockwise onto the syringe tip.

To avoid needle damage, remove the needle cover by pulling straight off. Do no twist when removing.

Use once and discard in accordance with local safety standards. Do not store at extreme temperatures and humidity.

Packaging: Regular wall needles.